




AB GERMA®

HEAVY DUTY TRANSFER MATTRESS

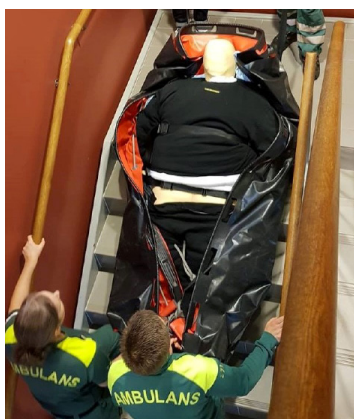

PRODUCT SHEET

Heavy Duty Transfer Mattress is developed to facilitate heavy duty patient handling. The mattress consists of durable and strong PVC coated polyester fabric material equipped with 6 carry handles on each long side plus evacuation handles on head and foot end of the mattress. This equipment is specifically designed to support and help and support the paramedics and rescuers to be able to transfer and transport a heavy duty patient. The construction and design make it possible to slide the mattress on the floor or down stair cases with patient onboard the mattress. When needed several rescuers are able lift and carry the patient over obstacles or on to a heavy duty stretcher by using the handles. In next step, from the stretcher to a hospital bed. The purpose is to support handling of heavy duty patient with safety and improved ergonomics.

PRODUCT SPECIFICATIONS

Approved according to European Standard EN 1865, CE-marked and X-ray and MRI translucent

PRODUCT INFO

Warranty period: 2 year

- 6 carry handles on each side make it easy to find balance for lifting when needed and carry a patient on board. Easy accessible handles up from ground lift.
- 2 handles on head and foot end to slide the patient on floors and staircases
- Sliding surface on the back of the Transfer Mattress.
- Easy to clean fully welded product
- 2 Safety belts to secure the patient and cover sheet to protect and hold the patient
- Durable and strong material and design with a carry capacity of 380 kg.
- Transfer Mattress should be accessible when needed.

MATERIAL

Material: PVC—polyester fabric material (flame retardant)

Rope: Nylon

Foam: 60 mm, Polyurethane foam

HEAVY DUTY TRANSFER MATTRESS



SPECIFICATIONS SIZE:

Length: 205 cm

Width: 143 cm

HEIGHT: 70 mm

WEIGHT: 8 kg

ORDER INFORMATION:

Art: Heavy-Duty Transfer Mattress

Art no: 271 050 000



Visiting address
AB Germa
Industrig. 54-56
SE-291 36 Kristianstad, Sweden

Postal address
AB Germa
Box 2127
SE-291 02 Kristianstad, Sweden

www.germa.se
Phone: +46 (0)44 12 30 30
info@germa.se

BENEFITS

Compliant to EN 1865

One-piece back, providing the patient with even spine support

Also allows preliminary X-ray diagnostics, direct in to x-ray no extra handling of patient.

Germa comfort technology gives rescuers quick handling and excellent patient ensuring safety is evenly dispersed to give the patient stable and optimal comfort.

Handles ergonomic grip, making patient lifts as comfortable as possible for rescuers

Handles are reinforced so a patient can be lifted in a patient lifting device.

Handles fixed to the bottom of mattress maintain stable when lifting the patient.

Robust welding—fully welded without any open seams or textiles, making hygienic cleaning of the mattress easy.

Outer material is made of flame retardant PVC-coated fabric It is flexible making it easy to mould the mattress around the patient.

Extra layer built under the mattress, safety, minimize risk of damage the fixation, last longer.

High load capacity 380 kg, safe to handle heavy duty people.

Closing mechanisms of the straps, plastic buckle Fix lock, allowing the user to quick tighten and adjust restraints on the patient.

HEAVY DUTY TRANSFER MATTRESS

PVC coated fabric makes it easy to clean and disinfect with most common disinfectants, easy to keep clean and ready for use.

Repair Kit easy to use for effective repair, quick to repair no return needed.

Visiting address
AB Germa
Industrig. 54-56
SE-291 36 Kristianstad, Sweden

Postal address
AB Germa
Box 2127
SE-291 02 Kristianstad, Sweden

www.germa.se
Phone: +46 (0)44 12 30 30
info@germa.se