DIRECTIONS FOR USE

STEP by STEP EASY TRANSFER

English: Directions for use. **Step by Step EasyTransfer**
Svenska: Bruksanvisning. **Step by Step Easy Transfer**

Distributor:

Ferno Washington Inc.
70 Weil Way
Wilmington, OH 45177
USA

Phone: +1 937 382 1451
Fax: +1 937 382 6569

E-mail: info@ferno.com

www.ferno.com

Manufacturer:

AB Germa
Industrigatan 54-56
SE-291 36 Kristianstad
Sweden

Phone: +46 (0)44 12 30 30
Fax: +46 (0)44 10 31 79

E-mail: info@germa.se

www.germa.se

12441920202 2015-10-25 version 1

English: These directions for use may be updated without further notice.
Copies of the current version are available from the manufacturer.

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1. Warning and caution statements

CAUTION

- Avoid handling Step by Step next to sharp objects.
- Avoid handling Step by Step next to open fire.
- On a current basis check that Step by Step webbing and seams are without damages from wear and tear.
- Not for use in CT, MRI or X-ray diagnostic equipment.

2. Intended use


Step by Step Easy Transfer is intended to be used as a tool for patient management from the accident scene to the stretcher in cooperation with Easy Transfer Mattress, Patient handling during evacuation to a stretcher for ambulance transportation. (EN 1865 and EN 1789).

Step by Step can be handled by two persons for a shorter distance, and a longer distance if two or more persons use additional straps. You can pull the mattress on the ground by assembling Step by Step as described in this manual on the designated handles.

2.a Tensile strength

Compilation of the tensile load of straps that are intended as aids when the stretcher is used for patient handling.

Picture of the strapped part under full (maximum) load, more than 250kg.

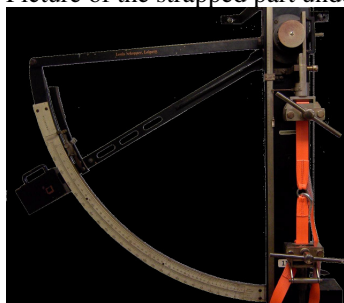3. Specifications

Step by Step Easy Transfer is in conformity with Council Directive 93/42/EEC concerning Medical Devices and EN 1865, EN 1789.

Temperature range: -30 °C to +70 °C

Parts/Materials

Material: Polyester webbing, coated polyester fabric
Carabiner: Steel galvanized

Dimensions	Length (cm)	Width (cm)	Height (cm)	Weight (kg)	N °of handles
Step by Step Easy Transfer	30	10	10	1	4

4. Description

Step by Step Easy Transfer is equipped with 2 handles at each side foot end head end.

Step by Step consists of two safety belts for securing the patient plus foot end strap connected to a foot bag and head strap. Materials are polyester safety belts plus control straps plus metallic carabiners and bag.

The Carrying Kit is stored in a practical bag that takes up a minimal amount of space.

5. Operating instructions

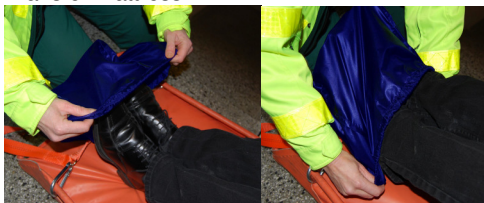
1. Open the bag take out the head strap and two safety belts from the bag.
2. Start by fastening the orange strap without a bag on the top handles on the Emergency Easy Transfer Mattress. Insure yourself that the snap hooks, straps and handles are in good condition, no damage or wear and tear.

3. Then you fasten the two blue patient safety belt on the side handles on a suitable place, the top strap should be as close to the arm crease as possible. The lower safety belt is appropriate to fasten high on the thigh but under the hip.

4. The lower orange strap with the bag should be fastened on the bottom handles on the Easy Transfer Mattress

5. Put the patients feet in the bag and then pull the lace lock above the ankles so that the feet won't slip out, but not that hard that there is a risk for stasis.

6. When the patient is secured on the Easy Transfer Mattress the transport can begin

7. Both operators should hold the orange straps with a firm grip (adjust the grip and position for the best ergonomic position). The operator on the foot end backs down at the same time as he holds the mattress by lifting lightly.

8. The operator on the head end holds against and pull carefully at the same time that he lightly lifts to prevent patient's head to hit against the steps. Throughout the entire maneuver it is very important to inform the patient what is occurring.

9. In a stair landing you use the Easy Transfer Mattress's bottom slip surface and drag the Emergency Mattress through the curve by pulling in the Step by Step straps using the handle.

10. Disconnect the Step by Step control straps before lifting the patient and mattress on to the stretcher.

6. Cleaning and Disinfecting

At a regular basis, Step by Step Easy Transfer should be wiped with a moist cloth and dried, if dirty, Step by Step may be washed with soapy water and rinsed with clean water. When needed the safety belts and straps can be washed in a washing bag in a washing machine. Note, mounting of carbiners when using washing machine.

Allow Step by Step to dry before storage.

In needs disinfection first clean Step by Step carefully and let it dry. Pour a solution containing 70 % alcohol or the disinfectant called LYSETOL AF on a clean cloth.

7. Repair of Step by Step Easy Transfer

If damage occurs repair as follows:

- a) Damage of carabiner – exchange the carabiner to a new one from us.
- b) Damage or wear and tear of patient safety belts exchange the belt from us.
- c) Wear and tear of any safety belt or straps shall be exchanged for safety reason.

8. Storage

Step by Step Easy Transfer to be stored in its bag in normal ambulance environment.

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Report no. :

(To be assigned in by
Manufacturer)

Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.

For claims please specify :

Purchase Date

Description of product

Description of Incident/Problem: 1) What, 2) How?

When? In connection with: 1) transport, 2) unpacking, 3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly, 10) storage, 11) other:

Info on the product, prior to problem,

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

Used cleaning and disinfecting agents/Sterilizing method:

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

Which **actions** have been: a) taken, b) planned by the reporter or user at the site of incident?

1) examination of the product on site, 2) replacement of the product, 3) other:

Suggested actions by the distributor/complainant

Returning of samples/faulty product(s),

- 1) Already shipped/Will be shipped to _____,
- 2) Pending request from manufacturer,
- 3) Product/parts not available:

Date:

We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.

Your contact person in this matter is: _____

Name:

Direct Tel.

Direct Fax.

Init/Date: